HYGISUN®

REF HS1016E

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PRECAUTIONS:

1. Please check the integrity of the packaging before use. Do not use the inner packaging bag if it is damaged or has foreign objects.
2. After pinching the soft liquid pack and shaking the ice evenly, please use it immediately, with a precise temperature control with a lowest temperature of 4°C. There is no risk of frostbite in clinical tests, but the human body has individual differences. If the cold compress feels too cold, stop using it immediately. If you have a sterile dressing, you can add a dressing layer and use it.
3. This product is self-cooling at first use, with a precise temperature control with a lowest temperature of 4°C. There is no risk of frostbite in clinical tests, but the human body has individual differences. If the cold compress feels too cold, stop using it immediately. If you have a sterile dressing, you can add a dressing layer and use it.
4. This product is only for external use. If the packaging is accidentally damaged and the inner liquid has gotten on your eyes or skin, please wash it off with clean water immediately.
5. When reusing this product, please check whether the interior is intact. Do not use it if there is leakage.

STORAGE:

Please place it in a well-ventilated indoor environment away from direct sunlight.

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How to use:

1. Activate the inner packaging bag by foldind or squeezing the liquid bag to soften it.
2. Pinch it up and down to make it mix well, open the inner bag and take out the cold compress.
3. Use Velcro to fix the product on the part that needs cold compress or follow the doctor's advice.
4. After 20-30 minutes of cold compress, if you need to use again, put the product into the inner bag and put it into the freezer.

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REUSE METHOD:

1. After freezing the cold compress for 1 hour, open the inner bag, take out the cold compress and repeat steps 3-4 above to use again.
2. If the cold compress is not used for a long time, please put it in the refrigerator to avoid freezing.

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Sichuan AI Doctor Medical Technology Co., Ltd.
No.333, Yongke Town, Wenjiang District, Chengdu City,
Sichuan Province, China

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4 260676 530850

DECLARATION OF CONFORMITY

ACCORDING TO (EU) 2017/745 MEDICAL DEVICE REGULATION

EU Representative

SUNGO Cert GmbH
Harffstr. 47, 40591 Düsseldorf, Germany
Contact: Frank Xu (Mr.)
SRN: NL-AR-000000247

Conformity Assessment

Conformity Assessment Procedure
Annex II+III of Regulation (EU) 2017/745

Applicable Standards
EN ISO 14971: 2019
EN ISO 15223-1: 2016
EN 1041:2008+A1:2013
ISO 10993-1: 2018
EN ISO 10993-5: 2009
EN ISO 10993-10: 2013

Remark

The declaration of conformity is valid in connection with the release technical document CE/MDR-M9001-01.

All the supporting documentation is retained at the premises of the manufacturer.

The Declaration of Conformity is exclusively under the sole responsibility of the manufacturer.

Manufacturer

Name: **SICHUAN AI DOCTOR MEDICAL TECHNOLOGY CO., LTD.**

Address: No.2000, Section 3, Wenquan Avenue, Chengdu Cross-strait Science and Technology Industry Development Park, Wenjiang District, 611130 Chengdu City, Sichuan Province, P.R. China.

Product Information

Name: **MEDICAL COOLING PATCH**

Model: HS1017E, HS1016E, KQ-I, KQ-II, KQ-III, KQ-IV, ET-I, ET-II, ET-III, ET-IV, BQ-I

GMDN: M9001

Basic UDI-DI: 697161168 patch001V9

UDI-DI:

(01)06971611685808 (11)211020 (17)231019 (10)20211020

Classification: Class I, According to Rule 1, Annex VIII, Regulation (EU) 2017/745

Declaration

We herewith declare that the above-mentioned products meet the requirements of Medical Device Regulation (EU) 2017/745 and the applicable standards above.

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