# HYGISUN<sup>®</sup>

REF HS1017E

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### Medical cooling patch Range of application

Fieber senken, abkühlen,  
Hitze lindern  
Reduces fever, cools down,  
relieves heat

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Wiederholtes Kühlpflaster  
Repetitive cooling patch

Medical cooling patch  
Range of application

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**Sichuan Ai Doctor Medical Technology Co., Ltd.**  
No.333, Yongke Town, Wenjiang District, Chengdu City,  
Sichuan Province, China

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#### How to use:

1. Activate the inner packaging bag by folding or squeezing the liquid bag to soften it.
2. Pinch it up and down to make it mix well, open the inner bag and take out the cold compress.
3. Use Velcro to fix the product on the part that needs cold compress or follow the doctor's advice.
4. After 20-30 minutes of cold compress, if you need to use again, put the product into the inner bag and put it into the freezer.

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#### REUSE METHOD:

1. After freezing the cold compress for 1 hour, open the inner bag, take out the cold compress and repeat steps 3-4 above to use again.
2. If the cold compress is not used for a long time, please put it in the refrigerator to avoid freezing.

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**PRECAUTIONS:**

1. Please check the integrity of the packaging before use. Do not use the inner packaging bag if it is damaged or has foreign objects.
2. After pinching the soft liquid pack and shaking the ice evenly, please use it immediately.
3. This product is self-cooling at first use, with a precise temperature control with a lowest temperature of 4°C. There is no risk of frostbite in clinical tests, but the human body has individual differences. If the cold compress feels too cold, stop using it immediately. If you have a sterile dressing, you can add a dressing layer and use it.
4. This product is only for external use. If the packaging is accidentally damaged and the inner liquid has gotten on your eyes or skin, please wash it off with clean water immediately.
5. When reusing this product, please check whether the interior is intact. Do not use it if there is leakage.

**STORAGE:**

Please place it in a well-ventilated indoor environment away from direct sunlight.

**HYGISON**

**DECLARATION OF CONFORMITY**  
 ACCORDING TO (EU) 2017/745 MEDICAL DEVICE REGULATION

**EU Representative**

SUNGO Cert GmbH  
 Harffstr. 47,40591 Düsseldorf, Germany  
 Contact: Frank Xu (Mr.)  
 SRN:NL-AR-000000247

**Conformity Assessment**

Conformity Assessment Procedure  
 Annex II+III of Regulation (EU) 2017/745

**Applicable Standards**  
 EN ISO 14971: 2019  
 EN ISO 15223-1: 2016  
 EN 1041:2008+A1:2013  
 ISO 10993-1: 2018  
 EN ISO 10993-5: 2009  
 EN ISO 10993-10: 2013

**Remark**

*The declaration of conformity is valid in connection with the release technical document CE/MDR-M9001-01.*

*All the supporting documentation is retained at the premises of the manufacturer.*

*The Declaration of Conformity is exclusively under the sole responsibility of the manufacturer.*

**Manufacturer**

Name: SICHUAN AI DOCTOR MEDICAL TECHNOLOGY CO.,LTD.

Address: No.2000, Section 3, Wenquan Avenue, Chengdu Cross-strait Science and Technology Industry Development Park, Wenjiang District, 611130 Chengdu City, Sichuan Province, P.R. China.

**Product Information**

Name: MEDICAL COOLING PATCH

Model: HS1017E, HS1016E, KQ-I, KQ-II, KQ-III, KQ-IV, ET-I, ET-II, ET-III, ET-IV, BQ-I

GMDN: M9001

Basic UDI-DI: 697161168 patch001V9

UDI-DI:

(01)06971611685808 (11)211020 (17)231019 (10)202111020

Classification: Class I, According to Rule 1, Annex VIII, Regulation (EU) 2017/745

**Declaration**

We herewith declare that the above-mentioned products meet the requirements of Medical Device Regulation (EU) 2017/745 and the applicable standards above.

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